

Let's get you drooling
and on your way.



ShieldT3
Health
TARGET • TEST • TELL

Follow instructions

Don't eat, drink, chew gum or smoke within 60 mins of test—this can cause discoloration.

Take your time


POOL saliva in your mouth, then **DROOL** onto the funnel. Don't spit — it can add bubbles and mucus.

Check your work

Use a light source, or ask for help from site staff, to check your sample against these images.

This is what an A+ sample looks like:

- Clear drool
- Correct volume (Between 1 and 1.5 mL)
- Cap is on tight

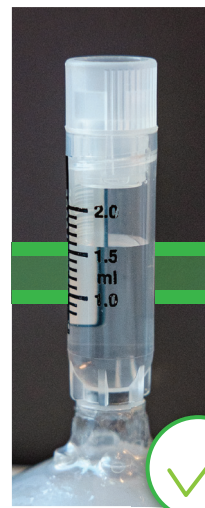

You will have to repeat the test if your tube looks like this:

Discoloration




Thick mucus




Glob of mucus



Particulates

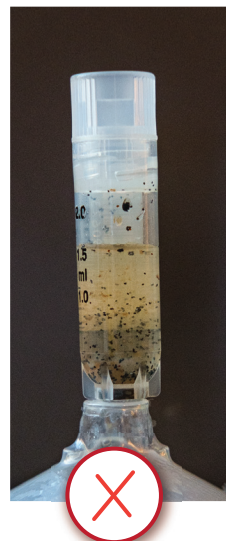

Wrong volume

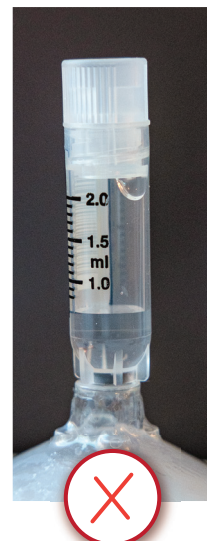

Leaky tube

